
READY TO WEAR PROTECTIVE SILICONE SKIN COVER

CARE INSTRUCTIONS

Congratulations on your new Ready-to-Wear protective silicone skin cover! A Step Ahead Prosthetics™ would like to thank you for choosing the SkinTones™ system.

THE SKINTONES SYSTEM provides protection to your prosthesis in addition to looking exceedingly natural. Our skins provide a strong and impermeable barrier between outside elements and your mechanical device. Before you go out and show off your new cover, please thoroughly read the following information to help maximize the longevity of your skin. If you have any questions or concerns, please visit the FAQ section of getskintones.com or contact us directly.

The outside surface of your protective silicone skin cover must be cleaned daily to avoid any buildup of foreign debris. Using a mild soap such as Dawn dish-washing liquid, thoroughly wash the outside of the protective silicone skin cover. Be sure to avoid letting water seep inside the prosthesis.

You may paint the toenails with nail polish – which should ONLY be removed with acetone-free nail polish remover. We recommend that you always bring your own supplies (polish and non-acetone nail polish remover) when going to a salon for your toenails.

Never walk without a solid-soled shoe or engage in high-impact sports or activities while wearing your protective silicone skin cover.

Some clothing dyes are not colorfast and will seep into the protective silicone skin cover, irreversibly staining it and making it impossible to fix. Wash or dry-clean new clothes before wearing them over the protective silicone skin cover. Some shoes, especially dyed leather, will have the same effect.

Never pull the silicone from any one point. If the silicone needs to be pulled up or adjusted for some reason, use as much surface area as possible, creating an evenly distributed amount of gentle pressure. If pulled or pinched in one area, the protective silicone skin cover may rip.

Never use self-tanners, lotions, creams, hand sanitizers or cosmetics of any kind on the protective silicone skin cover. Anti-bacterial soap should NEVER be used.

Never apply any product containing oil to the protective silicone skin cover as it can penetrate and weaken the silicone. If oil does make contact with the silicone, wash it down immediately with Dawn dish detergent mixed with water, or wipe it down with rubbing alcohol.

When properly installed and maintained, protective silicone skin covers should look great and last for years.